## OBLIČEJOVÝ ŠTÍT UCHYCENÝ NA NOSE

### Jak štít používat

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### Product specification:

- Protective helmet on the nose, fit for 10+ years person
- It is not a certified medical device
- Material used:
  - Front strip: PVC / EPDM laminate
  - Faceplate: PET 0.2 mm
- Regulating stoppers: polymer + plastic length regulator
- Made in Poland

## PROTECTIVE HELMET ON THE NOSE

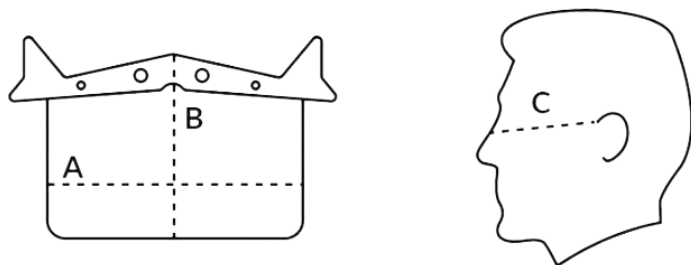
### How to use Protective helmet

- Before each use, disinfect the helmet with a suitable disinfectant containing at least 70% ethanol
- First attach the helmet loop behind one ear, then place it on the nose at the cut in the middle of the helmet. Finally place the second loop behind the second ear
- Adjust the drawstrings to the required length, the helmet needs to be slightly looser, so that it does not run up your nose, into your eyes. But it must not be too loose to drop down from face

### Helmet maintenance

- Before/after each use, disinfect the helmet with a suitable agent with a minimum of 70% ethanol content (for virucidal disinfection of the product). Do not use aggressive chemicals. that can damage the plastic structure
- Do not wash helmet in a washing machine, dry in a dryer, or in a microwave oven, etc.
- Do not bend the helmet aggressively, as this could break and cause injury

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Size / Rozměr	A	B	C
S	16 cm	10 cm	11 - 13 cm
M	17 cm	10 cm	13 - 15 cm
L	19 cm	11 cm	15 - 17 cm

In order to select the appropriate size of the mask, we recommend taking measurements directly on the face. Measure the "C" dimension marked in the picture with a tape measure or a string and check out the distance with a ruler.

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